

DCI Quick-Switch Water Bottle System

Looking for a quick & easy way to change water bottles? With the DCI Quick-Switch Bottle System simply push up and twist. Saves time and prevents bottle thread damage. Hygienists LOVE the Quick-Switch Water Bottle System.

- Prevents misalignment and cross threading of bottle and adaptor. This extends the life of the bottle and helps to prevent leaks.
- Saves time in changing bottles
- Expanded opening of bottle makes it easier to fill
- Safely releases pressure when removing bottle
- Easy installation requires no tools

Tech Tip for installation:

To install Quick Switch (QS) adaptor to bottle:

1. Apply a small bead of thread lock material on two full threads of QS adaptor.
2. Thread bottle into adaptor.
3. Let stand 30 minutes.

To replace bottle simply:

1. Remove QS adaptor using pliers or a strap wrench. Thread lock material is designed to allow removal of bottle from adaptor. (It is designed NOT to be permanent).
2. Remove any leftover thread lock from adapter before re-using.
3. Repeat installation steps.

Parts can be purchased as a kit or individually, depending on the needs of the office.

Kit – Converter and QS Adaptor **(8941)**

Converter (required for each water system) **(8942)**

QS Adaptor (required for each water bottle) **(8943)**

QS Adaptor package of 5 **(8944)**

Replacement sealing o-ring **(2297)**



Converter **(8942)**

QS Adaptor **(8943)**



Kit – Converter and QS Adaptor **(8941)**