

BEARING STRIP KIT FOR RIGID ARMS

Installation Instructions



Because post diameters vary significantly in the dental equipment industry, this bearing strip kit has been provided with your throughmounted arm.

If your through-mounted arm doesn't fit over the mounting post, or if it fits too tightly, you can remove the two black bearing strips in the through-mount and replace them with the pair of gray or white ones provided. The gray strips will provide an additional .010" clearance on your post; the white ones will provide an additional .020" clearance.

NOTE

The bearing strips must be used as pairs.
Do not mix colors.

REMOVE THE FACTORY-INSTALLED BEARING STRIPS

Use a knife blade or screwdriver to lift one end of the strip out of the channel in the through-mount (figure 1), and pull it free with your fingers. Be careful not to scratch the paint on the hub with your knife.

Remove the other strip the same way.

INSTALL THE GRAY BEARING STRIPS

This can be done with your fingers. Bend the strip into a circle and slide it into the channel in the through-mount (figure 2). Make sure the entire strip is seated in the channel.

Install the other gray strip the same way.

Reinstall the arm on the post. If the fit is still too tight, remove the gray strips and replace them with the white ones

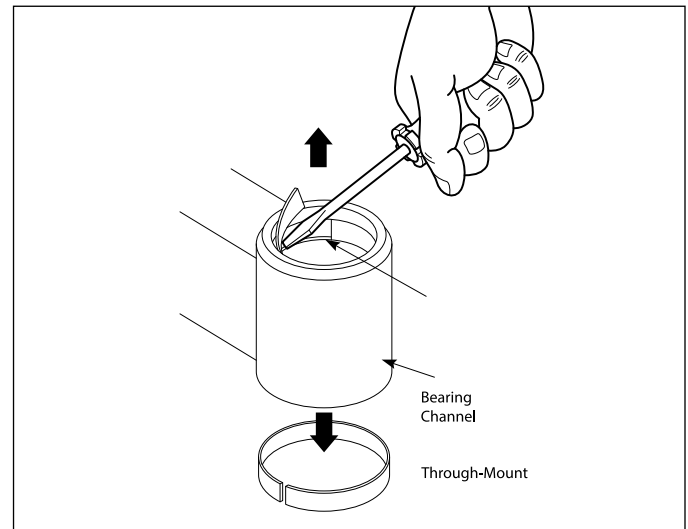


Figure 1 - Remove Strips

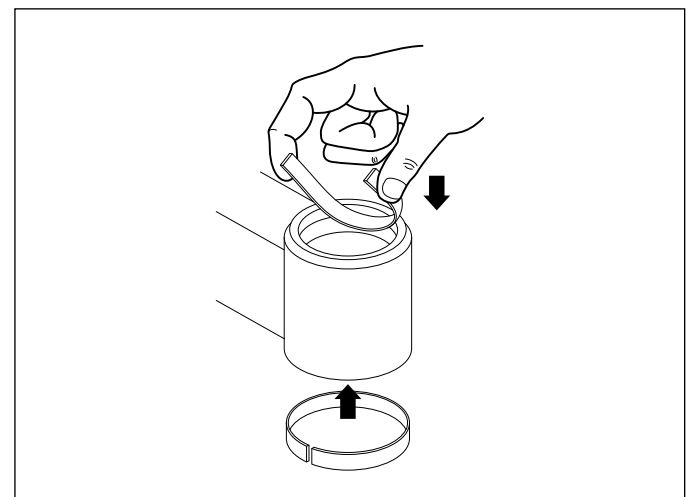


Figure 2 - Install Strips