

# QUICK INSTALLATION GUIDE


## Installation Instructions



The DCI 3 Position Deluxe Power Pack is preset to operate like any standard power optics light source system. The preset output voltage is 3.3 VDC (Setting #5) and the system is programmed to provide light when the foot control is activated. Program setting #9.

### For a new installation:

1. Turn the dental unit OFF and disconnect power.
2. Install a 1/8" X 1/8" X 1/16" tee fitting in the drive air line on the desired handpiece tubing.
3. Attach 1/8" tubing from the tee fitting to the proper handpiece position air signal input on the power optics source.
4. Install light source wiring to power pack.
5. Complete steps 2 & 3 with the remaining handpiece positions as necessary.
6. Set the voltage setting knob to 0 for unused handpiece positions.
7. Attach input power wires to the power optics source.
8. Turn the dental unit on and verify your new installation operates properly. Ensure a handpiece is installed to create back pressure to operate the power optics source.

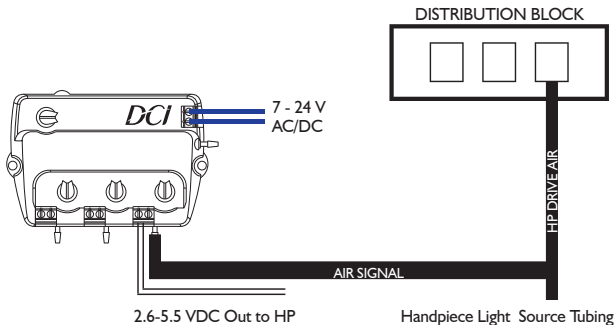

**FOR ADVANCED PROGRAM SETTINGS AND INSTALLATION OPTIONS,  
REFER TO THE USER INSTALLATION MANUAL 92518**

